

Suction Cannulas and Suction Sets Instructions for use

Ref. No.: 0202-.....

 <b style="font-size: 1.2em;">GRENA[®] <i>Think Medical</i> Grena (Qingdao) Medical Devices Ltd. , No 318 Huanghe West Road, Huangdao District, Qingdao City, Shandong Province, 266555, China	Contact information: Phone: +86 0532- 86769190 Fax: +86 0532- 86767322	<table border="1" style="margin: auto; border-collapse: collapse;"> <tr> <td style="padding: 2px 5px;">EC</td> <td style="padding: 2px 5px;">REP</td> </tr> </table> <p style="font-size: 0.8em; margin-top: 5px;"> MDML INTL Limited, 10 McCurtain Hill Clonakilty, Co. Cork Republic of Ireland </p>	EC	REP	
EC	REP				

Indications:

Suction cannulas and suction sets are used to draw liquids from the patient during surgery.

Contraindications:

Suction sets and suction cannulas should not be used if delicate tissue can be damaged by suction.

Product description:

Suction sets and suction cannulas are sterile and disposable. They are made of medical grade raw materials. Cannula can be easily disconnected from the suction tube and replaced by a new one if necessary.

Instructions for use:

1. Choose the appropriate size and type of suction cannula or suction set.
2. Open individual package using aseptic technique and place the device on the sterile surface.
3. Connect suction cannula with the connector of appropriate suction tube and check if connection is tight. If suction set is preassembled check connection tightness only.
4. Connect the other connector of suction tube to a suction source.
5. Set desired suction level.
6. Submerge suction cannula tip under fluid level to remove it from the surgical site.
7. If cannula is equipped with side hole, closing it with index finger starts suction while releasing stops suction.
8. Pool cannula consists of inner cannula and outer moveable perforated shield. Shield can be locked either in a proximal position exposing inner cannula tip for point suction or in a distal position for fluid evacuation by perforated surface of the shield.

Warnings and precautions measures:

1. Always check connections tightness prior to procedure.
2. Adjust suction to the level which allows avoiding delicate tissue damage.
3. Dispose of all opened suction cannulas or suction sets no matter if they were used or not.
4. Use immediately after opening.
5. The instrument requires appropriate disposal after use in accordance with all applicable local regulations including, without limitation, those pertaining to human health and safety and the environment.
6. This product is intended for single patient and procedure use. Resterilization, reuse, reprocessing, modification may lead to serious consequences.
7. Product is intended to be used exclusively by qualified medical staff.



Keep dry

Caution, consult
accompanying
documents



Consult instructions for
use



Do not resterilize



Manufacturer



Do not use if package is
damaged



Authorized
representative
in EU